

Accurate EtCO₂ monitoring with every breath


Microstream™ Advance filter lines have been proven in internal testing to deliver more accurate readings compared to Masimo NomoLine-O™^{†,†}

	Microstream™ Advance oral/nasal filter lines	Microstream™ Advance nasal filter lines	Masimo NomoLine-O™ [†] nasal/oral CO ₂ cannula with O ₂	Masimo NomoLine-O™ [†] adult nasal CO ₂ cannulas with O ₂
EtCO ₂ accuracy with supplemental oxygen	✓	✓	✗	✗
EtCO ₂ accuracy with varied respiratory rate	✓	✓	✓	✗
Microstream Advance™ filter lines are the only ones to deliver EtCO₂ accuracy with both supplemental oxygen and varied respiratory rate. ^{†,1}			Bench testing results showed Masimo NomoLine-O™ sampling lines had several readings with an error >2 mmHg as respiratory rate, supplemental oxygen flow, or both changed. ^{†,1}	

It's estimated that 80% of patients experience nasal cycling and closer to 86% of patients experience septal deviation.^{2,3}

Microstream™ Advance filter lines are the **ONLY** sampling lines with Uni-Junction™ sampling technology.⁵


Uni-junction™ sampling captures every breath. This patented technology is designed for effective sampling from both nares and the mouth, even at low tidal volumes leading to accuracy and clear waveforms.⁵

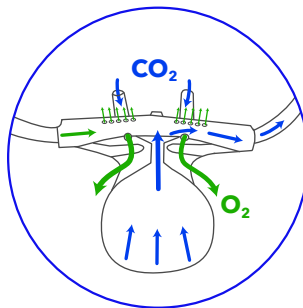

Masimo NomoLine-O™[†] uses a split-flow design.⁴

Split-flow cannula designs like Masimo NomoLine-O™[†] may lead to an **increase in sample dropouts and false alarms.**⁶

Masimo NomoLine-O™[†] IFU states:
"Do not use NomoLine-O™[†] cannulas on patients with obstructions in one or both nostrils."⁷

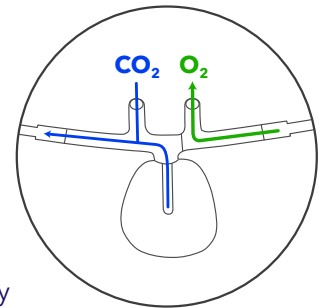

Microstream™ Advance filter lines are the only sampling lines validated to work with the Microstream™ capnography-enabled systems.

Using unvalidated capnography sampling lines on Microstream™ capnography modules may cause improper module performance⁸, result in inaccurate readings⁸, can affect patient and operator safety⁹ and may cause damage to the monitor or not meet claimed specifications.¹⁰

Masimo's NomoLine-O™ IFU states: **"This [Nomoline-O] warranty does not extend to any product that has been used in violation of the operating instructions supplied with the product."**^{7,†}

Microstream™ capnography is an integrated system engineered for delivering accurate results.¹¹



Accurate

The CO₂-specific technology called Molecular Correlation Spectroscopy™ (MCS) and 1.0mm ID microbore tubing from our Microstream™ Advance filter lines enable clear, **crisp waveforms and accurate measurements.**^{12,13}

Reliable

Proprietary Smart Capnography™ algorithms are included on all Microstream™ capnography-enabled monitors and work in the background to deliver a **more reliable respiratory rate.**¹⁴

Plug & Play

All Microstream™ capnography enabled monitors have a CO₂-specific technology. Because of this technology, Microstream™ capnography **does not require individual patient calibration.**¹⁵

† Please refer to the product IFU for all relevant indications, warnings, and complete details on usage.

The Microstream™ capnography monitoring system should not be used as the sole basis for diagnosis or therapy and is intended only as an adjunct in patient assessment.

1. Karpenkop I, Glzman D, Peri D, Liu KE. Not all cannulas are created equal: the influence of sampling line design on capnography performance. Medtronic White Paper. 11/2021.
2. Marshall SG, Henry NR, Russian, CJ. Right Versus Left Prong Nasal Cannula Flow Delivery and the Effects of Nasal Cycling on Inspired FiO₂ in an Adult Anatomic Model. Respiratory Care April 2016, Vol 61, No 4, 397-404.
3. Alghamdi FS, Albogami D, Alsurrayhi AS, et al. Nasal Septal Deviation: A Comprehensive Narrative Review. Cureus. 2022;14(11):e31317. Published 2022 Nov 10. doi:10.7759/cureus.31317
4. Masimo NomoLine-O™ Sampling Lines product information brochure. Accessed 12.16.24
5. Medtronic internal verification data on file.
6. Restrepo RD, Karpenkop I, Liu KE. Evaluation of capnography sampling line compatibility and accuracy when used with a portable capnography monitor. J Vis Exp. 2020;(163):e61670.
7. Masimo NomoLine-O™ sampling line Instructions for Use accessed 5.17.24
8. User manual-Alaris™ System with Guardrails Suite MX-2019-p.517
9. User manual-Connex™ Devices-Directions for Use-SW Version 2.4x-p. 16
10. Mindray PASSPORT 12/8-operator's manual-Section 27-Accessories
11. Maddox RR, Williams CK, Oglesby H et al. Clinical experience with patient-controlled analgesia using continuous respiratory monitoring and a smart infusion system. Am J Health Syst Pharm. 2006;63(2):157-164
12. MiniMediCO₂:Q005745 MINIMEDICO₂ V1 VERIFICATION REPORT. MicroMediCO₂:RE00142204 MicroMediCO₂ Carmel Verification Test Report. NanoMediCO₂:RE00214658 NanoMediCO₂ V2 System Verification 3rd cycle Test Report - Rise & Response Time
13. MiniMediCO₂: Q005856 N85 VERIFICATION REPORT. MicroMediCO₂: RE00184310MicroMediCO₂ Carmel Verification Report - Respiration Rate. NanoMediCO₂:RE00214659NanoMediCO₂ V2 System Verification 3rd Cycle Test Report - EtCO₂, FiCO₂, Respiration Rate
14. Michal E. Eisenberg, Dalia Givony, Raz Levin, Acoustic respiration rate and pulse oximetry-derived respiration rate: a clinical comparison study, Journal of Clinical Monitoring and Computing volume 34, pages 139-146 2020
15. MiniMediCO₂: Q005856 N-85 Verification Report. MicroMediCO₂: RE00142204 MicroMediCO₂ Carmel Verification Test Report. NanoMediCO₂: RE00157241 NanoMediCO₂ V2 System Verification Report - Operation Temp., Humidity & Auto Zero

©2025 Medtronic. Medtronic, Medtronic logo, and Engineering the extraordinary are trademarks of Medtronic.™ Third-party brands are trademarks of their respective owners. All other brands are trademarks of a Medtronic company. 02/2025 - US-PM-2400494 - [WF#13131228]

medtronic.com

Medtronic